<b>Blom-Singer</b> <sup>®</sup>
voice restoration systems
<p>37-401-01 Rev. A</p>

## Flushing Device Spülvorrichtung Dispositivo de lavado por descarga de agua Système de Vidange Dispositivo di lavaggio a getto Spoelapparaat Dispositivo de Descarga

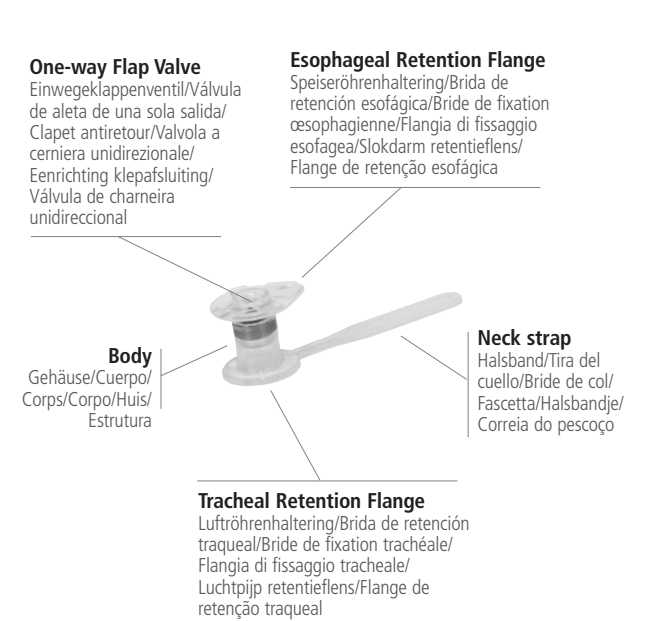
REF IN4050

**Flushing Device** Spülvorrichtung/Dispositivo de lavado por descarga de agua/Système de vidange/Dispositivo di lavaggio a getto/Spoelapparaat/Dispositivo de descarga

### Voice Prosthesis

Shunt-Ventil/Prótesis de voz/Prothèse vocale/Protesi fonatoria/ Stemprouthese/Prótese vocal

Repeat cleaning regimen as needed. The flushing device should be cleaned, rinsed and air-dried after each use. Fill and squeeze the flushing device bulb with 3% hydrogen peroxide after each use. Shake to remove any residual liquid. Allow the flushing device to dry in a well-lit, aerated area. If desired, the flushing device may be washed in soap and water, rinsed and air-dried.

Do not flush liquid through the voice prosthesis until the tip of the flushing device is properly inserted into the prosthesis, thus avoiding leaking water into the trachea. Should a small amount of water leak into the trachea, this will cause coughing.

*For users of the Blom-Singer Indwelling **Advantage**, please also refer to Indwelling **Advantage** user information data sheet.*

### ORDERING INFORMATION USA & Canada

Blom-Singer products can be ordered directly from International Healthcare Technologies (InHealth) at the following toll free number (800) 477-5969, Monday—Thursday, 6:30 am—4:00 pm; and Friday, 6:30 am—2:00 pm, Pacific Time. (The toll free number can be used from the continental USA, Alaska, Hawaii, Puerto Rico, and the Virgin Islands. The toll free number for Canada is (800) 461-0991.)Products also may be ordered at the InHealth website: www.inhealth.com

If you have any questions or dissatisfaction with the product please contact customer service via telephone, fax, post, or e-mail: Tel: (800) 477-5969; Fax: (888) 371-1530; Post: 1110 Mark Avenue, Carpinteria, CA 93013-2918; E-mail: order@inhealth.com

#### International

For a list of international distributors, please contact International Healthcare Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA. Tel: +01(805) 684-9337, Fax: +01(805) 684-8594.

#### RETURNED GOODS POLICY

All returned merchandise must have a Return Merchandise Authorization number (RMA), and must be unopened and undamaged. RMA numbers are obtained from InHealth Customer Service. Returns without an RMA will not be accepted. Products must be returned in unopened packages, with manufacturer’s tamper evident seals intact to be accepted for replacement or credit. Products will not be accepted for replacement or credit if they have been in possession of the customer for more than 90 days. There is a 20% restocking fee on all returned merchandise.

<p><b>INHEALTH</b><sup>®</sup></p> <p>TECHNOLOGIES</p>
<p>User Information Data/Informationsblatt für Benutzer/Hoja de información del Usuario/Fiche Technique d’Information Utilisateur/Scheda informativa per il paziente/Informatieblad voor gebruikers/Folheto Informativo do Utente</p>

#### INDICATIONS FOR USE

The Blom-Singer Flushing Device is intended for flushing small particulate matter from the inside of a Blom-Singer voice prosthesis that has been positioned in the tracheoesophageal puncture. *For users of the Blom-Singer Indwelling **Advantage** voice prosthesis, the flushing device is intended to be used in conjunction with the Blom-Singer Cleaning Brush. Please refer to the Indwelling **Advantage** user information data sheet for complete information.*

#### CONTRAINDICATIONS

The Blom-Singer Flushing Device is a medical product that should be used only by individuals with experience and training in its use and care.

#### HOW SUPPLIED

The Blom-Singer Flushing Device is supplied as a 3-pack, clean-and-ready to use.

#### ROUTINE CARE AND CLEANING

To keep the inside of the prosthesis body and valve clean of dried phlegm, it is recommended that the prosthesis be flushed, *carefully*, on a regular basis. Your hands and cleaning accessories should **always be thoroughly clean** to help avoid introducing contaminants into the tracheostoma, puncture, or esophagus. Flushing should only be done in front of a mirror with a bright light focused directly on the stoma so that the open end of the prosthesis is clearly visible. Before use, first rinse the inside of the flushing device with 3% hydrogen peroxide and then with water. Use long handled tweezers to carefully remove any dried phlegm that may be in the open end of the voice prosthesis.

#### Flushing

To fill the flushing device with water, squeeze the bulb and then release it in clean water. Shake excess water from the tip of the flushing device before flushing the prosthesis. Carefully and gently insert the tip of the flushing device into the prosthesis until it fits completely against the opening of the voice prosthesis (diagram 1). Do not flush water through the voice prosthesis until the tip of the flushing device is properly inserted into the prosthesis, thus avoiding leaking water into the trachea. Should a small amount of water leak into the trachea, this will cause coughing. Squeeze the bulb on the flushing device to flush a rapid squirt of water through the voice prosthesis. After flushing, remove the flushing device carefully.

#### Routine Care

Repeat cleaning regimen as needed. The flushing device should be cleaned, rinsed and air-dried after each use. Fill and squeeze the flushing device bulb with 3% hydrogen peroxide after each use. Shake to remove any residual liquid. Allow the flushing device to dry in a well-lit, aerated area. If desired, the flushing device may be washed in soap and water, rinsed and air-dried.

#### WARNINGS AND PRECAUTIONS

User’s hands and flushing device must be cleaned before use.

Never flush voice prosthesis with any substances other than clean water.

**Care should be exercised when inserting or removing cleaning devices to avoid accidental displacement of the prosthesis, which could result in inhalation of the voice prosthesis. In the unlikely event that this should occur, you should attempt to cough the voice prosthesis out of the windpipe and immediately seek medical attention if unsuccessful.**

If the voice prosthesis is accidentally dislodged from the puncture, **immediately** place a 22 Fr Blom-Singer tracheoesophageal puncture dilator in your puncture to keep the puncture from closing. Return to your clinician within 24 hours for reinsertion of a voice prosthesis.

If there are tears, cracks, or structural damage to the flushing device discontinue use.

Do not flush liquid through the voice prosthesis until the tip of the flushing device is properly inserted into the prosthesis, thus avoiding leaking water into the trachea. Should a small amount of water leak into the trachea, this will cause coughing.

*For users of the Blom-Singer Indwelling **Advantage**, please also refer to Indwelling **Advantage** user information data sheet.*

#### ORDERING INFORMATION USA & Canada

Blom-Singer products can be ordered directly from International Healthcare Technologies (InHealth) at the following toll free number (800) 477-5969, Monday—Thursday, 6:30 am—4:00 pm; and Friday, 6:30 am—2:00 pm, Pacific Time. (The toll free number can be used from the continental USA, Alaska, Hawaii, Puerto Rico, and the Virgin Islands. The toll free number for Canada is (800) 461-0991.)Products also may be ordered at the InHealth website: www.inhealth.com

If you have any questions or dissatisfaction with the product please contact customer service via telephone, fax, post, or e-mail: Tel: (800) 477-5969; Fax: (888) 371-1530; Post: 1110 Mark Avenue, Carpinteria, CA 93013-2918; E-mail: order@inhealth.com

#### International

For a list of international distributors, please contact International Healthcare Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA. Tel: +01(805) 684-9337, Fax: +01(805) 684-8594.

#### RETURNED GOODS POLICY

All returned merchandise must have a Return Merchandise Authorization number (RMA), and must be unopened and undamaged. RMA numbers are obtained from InHealth Customer Service. Returns without an RMA will not be accepted. Products must be returned in unopened packages, with manufacturer’s tamper evident seals intact to be accepted for replacement or credit. Products will not be accepted for replacement or credit if they have been in possession of the customer for more than 90 days. There is a 20% restocking fee on all returned merchandise.

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|--|

**BIBLIOGRAPHY, BIBLIOGRAPHIE, BIBLIOGRAFIA, BIBLIOGRAFIE**

- Blom, E.D.: "Tracheoesophageal Speech", Seminars in Speech and Language: Laryngectomee Rehabilitation, 16:3, Curlee, McFarlane, Watteson, (Eds), Thieme Medical Publishers, 1995.
- Blom, E.D., Hamaker, R.C.: "Tracheoesophageal Voice Restoration Following Total Laryngectomy", Cancer of the Head and Neck, (Third Edition) Myers, E.N. and Suen, J.Y., (Eds), W.B. Saunders, Philadelphia, 1996.
- Blom, E.D., Hamaker, R.C., Freeman, S.B.: "Postlaryngectomy Voice Restoration", Highlights of the Instructional Courses, Vol. 7, Lucente, F.E. (Ed.) Mosby, St. Louis, 1994.
- Blom, E.D., Pauloski, B.R., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part I: Speech Characteristics", Laryngoscope 105:10: 1093-1103, 1995.
- Blom, E.D., Singer, M.I. "Disinfection of Silicone Voice Prostheses", Arch Otolaryngol, 112: 1986.
- Blom, E.D., Singer M.I., and Hamaker, R.C. "A Prospective Study of Tracheoesophageal Speech", Arch Otolaryngol, 112: 440-447, 1986.
- Blom, E.D., Singer M.I., and Hamaker, R.C. "An Improved Esophageal Insufflation Test", Arch
- Hamaker, R.C., Singer M.I., Blom E.D. and Daniels, H.A. "Primary Voice Restoration at Laryngectomy", Arch Otolaryngol, 111: 182-186, 1985.
- Pauloski, B.R., Blom, E.D., Logemann, J.A., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part II: Swallow Characteristics", Laryngoscope 105:10: 1104-1110, 1995.
- Singer, M.I. and Blom, E.D. "An endoscopic Technique for Restoration of Voice After Laryngectomy", Ann Otol Rhinol Laryngol, 89:6: 529-533, 1980.
- Singer, M.I. and Blom, E.D. "A Selective Myotomy for Voice Restoration After Total Laryngectomy", Arch Otolaryngol, 107: 670-673, 1981.

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